

## SB22 – COVID testing comms / instructions


- It is the Austrian guidance that all attendees should take an LTF test every 24hrs
- These must be the Austrian LFT and not NHS tests; they are free and available from the Snowbombing HQ , the merch stand and the customer service desk.
- Your pack contains a set of 5 tests and QR codes that must be attached to each test cassette
- When taking your test please follow the instructions on the back of the box and within this document.
- Results will need to be uploaded to the government portal, via the QR code below – the website can be translated to English via the top right hand corner.
- If you test positive stay in your accommodation and contact the covid safety team on the snowbombing app
- You must show the test result to get into all SB operated venues.

Step by step instructions to upload your daily LFT.

- 1) Scan the QR code which will take you to the website where you are required to upload your LFT result.



- 2) Once you click on the link, you will need to register to upload your test result. The page looks like this.



COVID-19 Self-test

DE

EN

1

Register

\* Fields must be filled!

First name (and middle name(s), if app...

Last name \*

Date of birth \*

Austrian SSN

Optional. Skip if you don't have an Austrian SSN!

Street \*

House number (and door number, if ap...

Postal code and city \*

Mobile number \*

Make sure you select EN (English) at the top of the page.

This is your hotel or apartment address. You can get this from your accommodation reception.

AA

selbsttest.tirol



Notification via ☒ E-Mail ☐ SMS

Ensure you select email. Do not select SMS.

I hereby confirm that the data I have provided is correct

☐ and that the test procedure has been carried out correctly.

I agree that my data will be stored and processed for

☐ the purposes permitted by law and confirm that I have read the [data protection information](#).

Register

- 2 Confirm
- 3 Perform test
- 4 Upload test
- 5 Invalidate test
- 6 Receive certificate

Imprint: <https://www.tirol.gv.at/gesundheitsvorsorge/infekt/coronavirus-covid-19-informationen/tirol-testet/selbsttests/impressum/>

Privacy policy: <https://www.tirol.gv.at/gesundheitsvorsorge/infekt/coronavirus-covid-19-informationen/tirol-testet/selbsttests/datenschutz-covid-19-selbsttests/>


v1.3.0

AA

selbsttest.tirol



- 3) Once you have completed the registration form, you will receive an email from **COVID-19 Selbsttest**. Ensure you check your junk email.

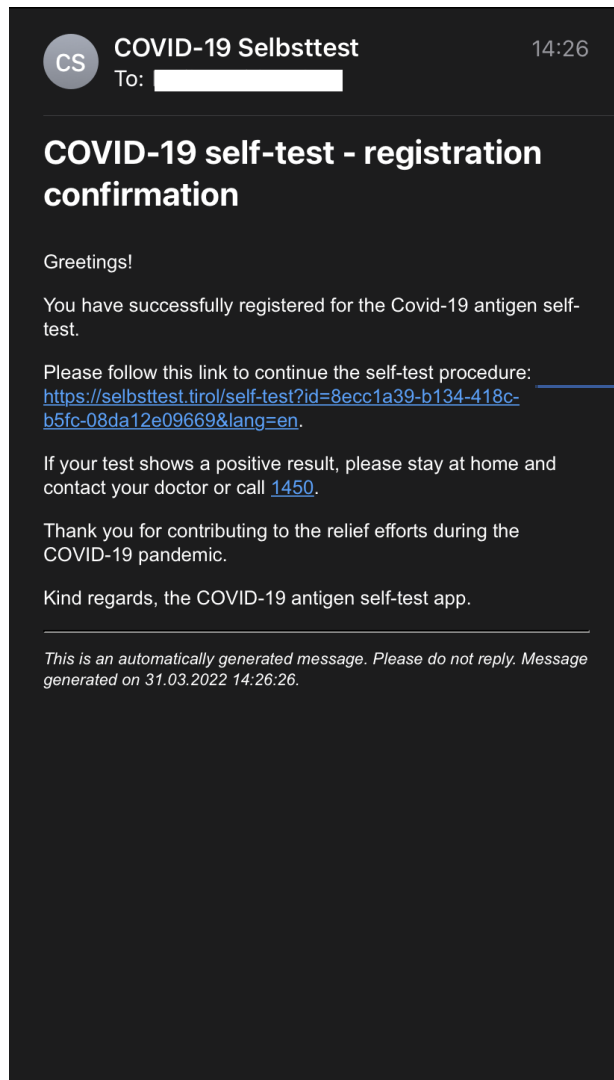

Click on this link. Here you will be able to upload your LFT result.

- 4) You will be then asked to enter your DOB to confirm your identity.  
5) You will be taken to a new page where you need to take your LFT and log your result.



## COVID-19 Self-test

✓ Register

✓ Confirm

3 Perform test

1. Please take the test kit out of the package.
2. Perform the test according to the [Instruction manual](#)
3. Make sure that the result is clearly visible.  
This can take up to 15 minutes.

Please specify your test result here:

The result is to be interpreted according to the [Instruction manual](#)



Positiv

POSITIVE 😞



Negativ

NEGATIVE 😊

4 Upload test

\*Before you take your test, ensure you have attached the stick on QR code to the top of the test. You will need this to log your result.

- 6) Once you have completed your test and logged your result, you will need to add the QR code sticker to the top of the test cassette.

✓ Confirm

✓ Perform test

4 Upload test

**Apply QR code sticker**

Apply the QR code sticker to the front of the test kit.

[Show example picture](#)

IMPORTANT: The result window must not be covered. The QR code is valid only once!

**Take a photo of the test kit**

The test kit with the QR code sticker must be photographed. How would you like to continue?

I would like to use my camera.


I have already taken a photo.

5 Invalidate test

6 Receive certificate



- 7) You will then need to take a picture of your completed test with QR code to provide your result certificate. Note you will need to take two (2) pictures of the completed test with QR code for it to log in the system.

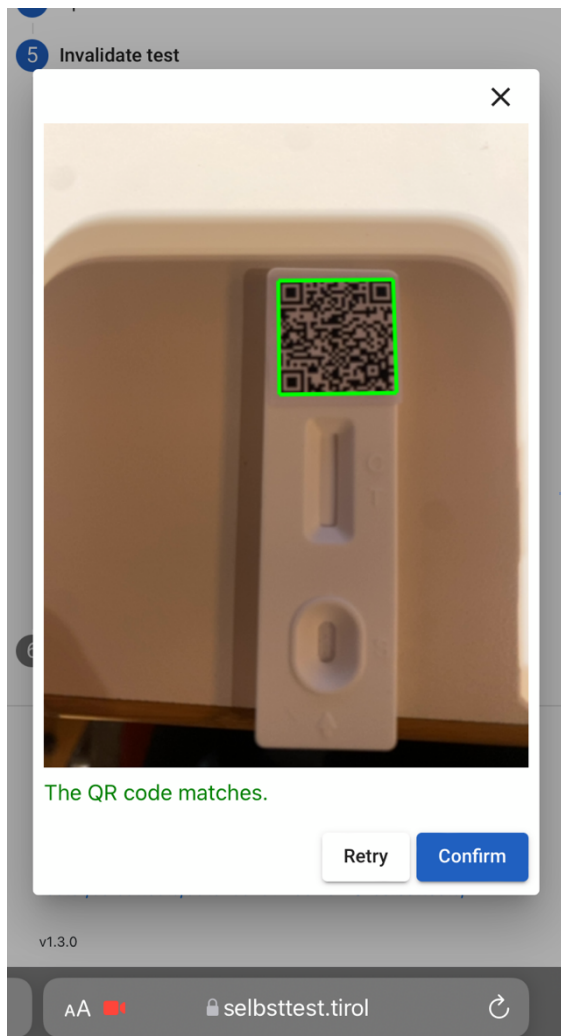

The picture will confirm when the QR code matches successfully.

- 8) Once your test result is registered, the web page will refresh to inform you its successful.





## COVID-19 Self-test

- ✓ Register
- ✓ Confirm
- ✓ Perform test
- ✓ Upload test
- ✓ Invalidate test
- 6 Receive certificate

### Your test result is ready!


We will soon send you a link per e-mail to the following address:

If you have not received the email, please also check the spam folder in your mailbox.

With this link you can retrieve your personal report at any time. The validity of this report depends on the purpose of use which can be retrieved under the same link.

You can now close this page.


- 
- 9) You will receive an email from **COVID-19 Selbsttest**. This will send you a link to download your LFT certificate. Ensure you check your junk email folder.

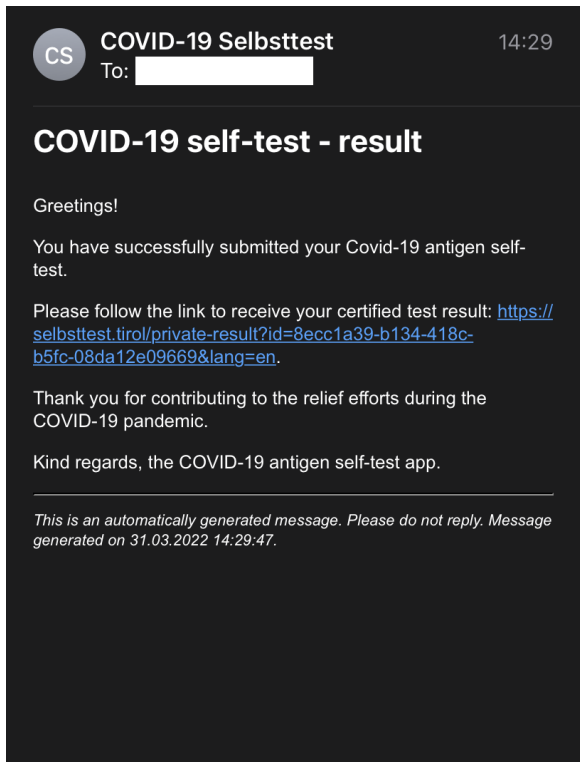

Click on this link. Here you will be able to download your LFT certificate.

- 10) Click on the link where you will be taken to a web page where you can view your verified result.

Result:  
**NOT DETECTED**

Download PDF 

Ensure you download PDF &  
save result to your phone and  
show it at the entrances.

First name:

Last name:

Date of birth:

Test performed on: 31.3.2022 14:27

Venue entrance test 

Valid until 1.4.2022 14:27

expires in 24 hours





## ERGEBNISPROTOKOLL

Dieses Schreiben protokolliert einen SARS-CoV-2-Test im Rahmen der Aktion "Selbsttest Tirol".

### GETESTETE PERSON:

Herr/Frau:

Geburtsdatum:

Test-Typ: Antigen-Test auf SARS-CoV-2

Getestet am: 31.3.2022 14:27

Gültig bis: 1.4.2022 14:27

Zum Zeitpunkt der Probenentnahme lautet das Testergebnis:

**NICHT NACHGEWIESEN**



Dieser Nachweis ist ausschließlich durch die darin angeführte Person für die in den jeweiligen Verordnungen festgelegten Zwecke zu verwenden. Eine missbräuchliche Verwendung oder Verfälschung des Nachweises kann strafrechtliche Konsequenzen zur Folge haben.

The above picture shows the downloadable PDF test result.